

TEST INCLUDES

Drug Screen

- 6-Acetylmorphine
- Amphetamine
- Barbiturate
- Benzodiazepine
- Cannabinoid
- Cocaine
- EDDP Specific
- Fentanyl
- Opiate
- Oxycodone
- Phencyclidine
- Specific Gravity
- Oxidant
- pH

Reflex Confirmation

- Amphetamines
- Benzodiazepines/Sedatives
- Illicit
- Opiates
- Oxycodones

Full Confirmation

- | | | |
|--|---|---|
| <ul style="list-style-type: none"> • 6-Acetylmorphine • 7-Aminoclonazepam • Alpha-PVP • Alprazolam • Amitriptyline • Amphetamine • Benzoylcegonine • Buprenorphine • Carisoprodol • Citalopram • Clonazepam • Clozapine • Codeine • Cotinine • Cyclobenzaprine • Desipramine • Dextromethorphan • Dextrorphan • Diazepam • Diphenhydramine • Doxepin • EDDP • Fentanyl • Flunitrazepam | <ul style="list-style-type: none"> • Fluoxetine • Flurazepam • Gabapentin • Hydrocodone • Hydromorphone • Imipramine • Ketamine • MDA • MDEA • MDMA • Meperidine • Mephedrone • Meprobamate • Methadone • Methamphetamine • Methylone • Methylphenidate • Morphine • Naloxone • Naltrexone • Norbuprenorphine • Norfentanyl • Norhydrocodone • Noroxycodone | <ul style="list-style-type: none"> • Noroxymorphone • Nortriptyline • O-Desmethyl-cis-tramadol • Oxazepam • Oxycodone • Oxymorphone • Pentazocine • Phencyclidine • Phentermine • Pregabalin • Propoxyphene • Risperidone • Ritalinic Acid • Sertraline • Tapentadol • Temazepam • Tramadol • Trazodone • Triazolam • Zaleplon • Zolpidem • Zolpidem Phenyl-4-Carboxylic Acid |
|--|---|---|

EXPECTED TURN AROUND TIME

1-2 Days

SPECIMEN

Urine

VOLUME

Min. 30 mL

CONTAINER

Urine Cup & Vacuette



COLLECTION

- Ensure all proper Personal Protective Equipment measures are taken.
- Provide the patient with a clean collection cup and instruct the patient to fill container at least half full (a minimum of 30 mL).
- Upon receipt of specimen from patient, record temperature. NOTE: Measure temp. within 4 min. of collection. It should read between 90°F to 100°F. If not within this range discard/recollect
- Label one vacuette tube with Patient name (first and last) and DOB
- Continue with transfer from cup to vacuette
- Remove seal from the top of the collection cup lid
- Invert labeled tub onto the spike on the cup
- Fill tube completely or until it will no longer fill
- Clean cup and vacuette and place in transport bag along with requisition w/ ICD-10, face sheet, medication list and insurance card front/back

STORAGE/TRANSPORT INSTRUCTIONS

- At room temp 15°C to 30°C for 7 days before analysis
- Refrigerated 2°C to 8°C for 14 days before analysis
- Frozen -15°C to -20°C for 30 days before analysis

STABILITY

5 Days

REJECTION REASONS

- Incorrect, improper or incomplete patient sample information
- Patient information on specimen label, requisition or patient demographics do not all match
- Insufficient sample volume: In certain circumstances when the required volume is not met or it is extremely difficult or impossible to recollect, the ordering physician may request the sample to be tested.
- Anything other than urine is present in the specimen container.